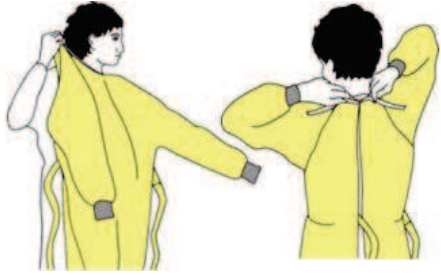


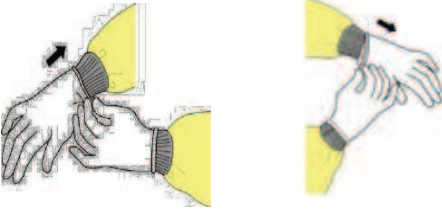
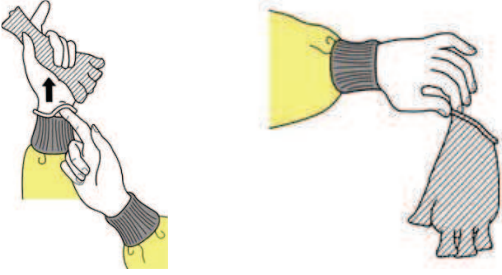

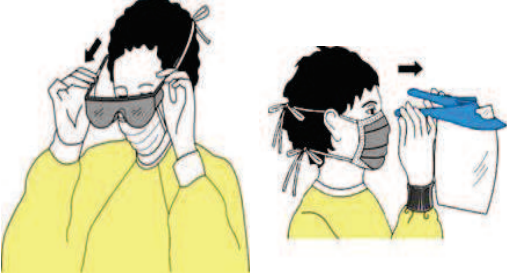



Personal Protective Equipment Sequence for Donning and Doffing

Sequence for donning PPE

Perform Hand Hygiene	
<p>GOWN</p> <ul style="list-style-type: none"> Select a long sleeved fluid repellent gown The gown must completely cover the torso from neck to knees, arms to end of wrists, and wrap around the back Secure ties at back of neck and side of waist If the gown is too small, use two gowns – gown one tie at the front, gown two ties at the back. 	
<p>FLUID REPELLANT MASK OR P2(N95) RESPIRATOR</p> <ul style="list-style-type: none"> Choose appropriate mask Ensure correct mask size is selected * Place over nose, mouth and chin Secure on head with ties or elastic bands Fit the flexible band over nose bridge Fit snug to face and below chin <p>If mask is damaged, soiled or damp, leave Room, perform hand hygiene and change mask</p> <p><i>*P2 (N95) respirator N95 key points;</i></p> <ul style="list-style-type: none"> Two sizes available – small and regular Perform fit-check each time a new mask is worn. Do not re-adjust mask once in patient's room 	
<p>PROTECTIVE EYE WEAR OR FULL FACE SHIELD</p> <ul style="list-style-type: none"> Position over face and eyes and secure to the head Adjust to fit comfortably. 	
<p>GLOVES</p> <ul style="list-style-type: none"> Select correct glove type and size <p>Extend gloves to cover gown cuffs.</p>	

Sequence for doffing PPE

<p>GLOVES</p> <ul style="list-style-type: none"> • The outside surface of gloves are considered contaminated • Grasp outside of glove with opposite gloved hand; peel off • Put removed glove in gloved hand • Slide fingers of ungloved hand under remaining glove at wrist. • Peel glove off from the inside over first glove <p>Discard gloves into clinical waste bin.</p>	
<p>Perform Hand Hygiene</p>	
<p>GOWN</p> <ul style="list-style-type: none"> • Gown front and sleeves are considered contaminated • Unfasten ties • Peel gown away from neck and shoulders, touching inside of gown only • Turn contaminated outside of the gown toward the inside • Fold or roll into a bundle <p>Discard gown into clinical waste bin</p>	
<p>Perform Hand Hygiene</p>	
<p>PROTECTIVE EYE WEAR OR FACE SHIELD</p> <ul style="list-style-type: none"> • The outside of protective eye wear/face shield are considered contaminated. • To remove, use only handle by head band or ear pieces • Discard disposable full face shield into clinical waste bin. • Reusable protective eye wear to be cleaned with a 2 in 1 detergent disinfectant wipe. 	
<p>Perform Hand Hygiene</p>	
<p>MASK / RESPIRATOR</p> <ul style="list-style-type: none"> • Front of mask/respirator is contaminated – DO NOT TOUCH • Grasp the bottom, then top tie/elastics and remove <p>Discard mask/respirator into clinical waste bin</p>	
<p>Perform Hand Hygiene</p>	