

EMIP - IntelePACS

InteleConnect Set-up Guide for Health Practitioners

- **InteleConnect** can be found under the **Health Apps** folder under the Windows Start menu. It is a web-based application that will open in Edge.



- **Alternatively:** Navigate to <https://inteleconnect.hdwa.health.wa.gov.au/> in Edge.
- Log-in with your **he-number** and **Windows password**.

Note: If on a shared workstation log-in **DO NOT** allow Edge to save the password.

For access issues or password resets, please contact **HSS ICT Support** on **13 44 77**

Navigation and Interface

Welcome Reg Ris-Karisma | My notifications | Critical | Patients | Profile | Help | ↗ Log out

Home Navigation		
Icon	Function	Description
<u>My notifications</u>	Notifications Tab	Will be used to display Order Status updates for your patients post Go-Live. Note: Your full name must be written on request forms for this to function.
<u>Critical</u>	Critical Notifications Tab	Will be used display Critical Result: Failsafe alerts for your patients post Go-Live.
<u>Patients</u>	Patient List Tab	Search for patients or create dynamic Worklists to track patients matching certain criteria.
<u>Profile</u>	Profile Tab	Make changes to your notification settings. Note: Contact details can only be updated via your HealthPoint profile.
<u>Help</u>	Help Button	Click the Help button on any window to bring up contextual help information for the screen you are currently viewing.
↗ Log out	Log Out Button	Click Log Out to end your session.

Our purpose: We support our customers to provide excellent health care

Patients Screen

Patient Search

Search using patient name or UMRN. Use the drop-down menu to include a date-of-birth.

External Archive

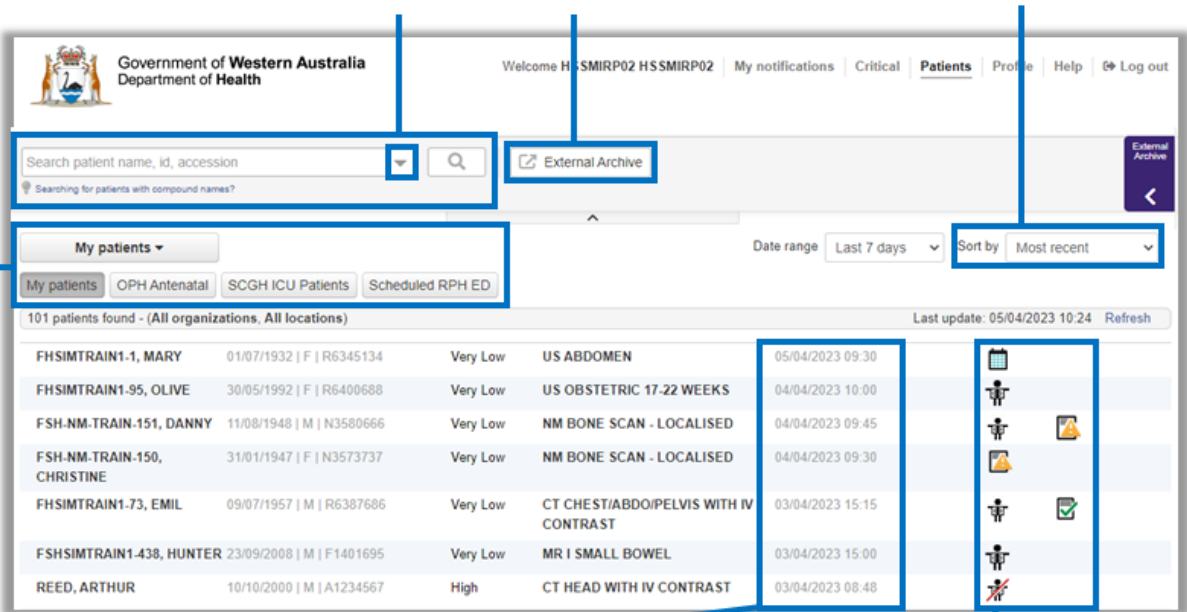
Search for images obtained at certain private radiology clinics.

Sorting

Adjust sort order of the current Worklist.

Worklists

Customised lists based on parameters you have chosen.



Date/Time

Scheduled exams will display the scheduled date and time of the exam. Once imaging is completed, the completed date and time will be displayed.

Order Status

Icons indicate the status of a patient's imaging / reporting. Hover mouse over icons for a description.

Order Status

Order Status Icons		
Icon	Code	Description
	SC	Scheduled: The order has been booked for imaging, or, the patient has presented to the Radiology department.
	IP	Patient Arrived / In Procedure: The examination has commenced in Medical Imaging.
	CM	Completed: Imaging for the study has been completed with images available to view on PACS (IntelConnect / IntelViewer).
	ZA	Dictated: Report drafted. Audio <u>not</u> available for review.
	ZY	Preliminary Report: A preliminary report is available (will be pictured alongside icon)
	CA	Cancelled: The request has been Cancelled by Radiology.
	HD	On Hold. The request has been Received by Radiology.
	ZZ	Report Available: A final report is available.
		A Critical Result: Failsafe Alert has been identified and requires action and acknowledgement.
		A Critical Result: Failsafe Alert has been acknowledged.

Our purpose: We support our customers to provide excellent health care

Patient and Exam View

The screenshot shows the IntelConnect EV interface. On the left, there is a search bar and a 'My patients' dropdown menu. Below it, a list of search results is shown, with 'Battle, Howard Max' selected. On the right, the patient's details are displayed, including DOB (1935-May-19), Gender (Male), and Patient ID (1560685478). Below the patient details, there are buttons for 'Launch IntelViewer', 'Grant Access...', and 'Add Impression...'. The main area shows exam details for 'CR CHEST 2 VIEWS FRONTAL AND LATERAL', including Priority (Unknown), Accession Number (1835412746), Referring Physician (N/A), Organization (IMS), Status (Unknown), Study Date (2004-Jul-12 09:22 AM), Radiologist (Unknown Radiologist), and Location. Two thumbnail images are shown: 'CR PA (1 image)' and 'CR LAT (1 image)'.

1. Open the order that you want to view.

2. Click a thumbnail to view the images in IntelConnect Enhanced Viewer.

Imaging View

The screenshot shows the IntelConnect Enhanced Viewer interface. At the top, there is a toolbar with various icons. Below the toolbar, there is a list of studies for 'BATTLE, Howard Max (71Y M)'. The list includes studies from Feb 13, 2007, Dec 12, 2006, Aug 23, 2006, Jul 03, 2006, Jun 15, 2006, Jun 08, 2006, and Jun 08, 2006. A yellow star icon indicates the current study. A gray outline indicates a series in an unselected viewport. Prior studies are distinguished with a color scheme. A red arrow points to a thumbnail in the list. Below the list, there are two viewports showing chest X-rays. A yellow outline indicates the selected viewport. The viewports show patient details: 'Anonymized InstitutionName BATTLE, Howard Max M, 71Y DOB: May 19, 1935 PID: 1560685478' and 'InstitutionName Howard Max M, 71Y May 19, 1935 1560685478'. The viewports also show 'Wt: 4096 L: 2048'.

A ★ indicates the current study

Gray outline indicates a series in an unselected viewport

Prior studies are distinguished with a color scheme

Prior studies list
Click the pin icon to pin the study to the thumbnail strip

Show or hide the viewport text overlay (O)

To view images, drag images from the thumbnail strip to the viewports

To navigate through a study, press the **Right Arrow/Left Arrow** key, or move the mouse wheel down/up, to view the next/previous page of series


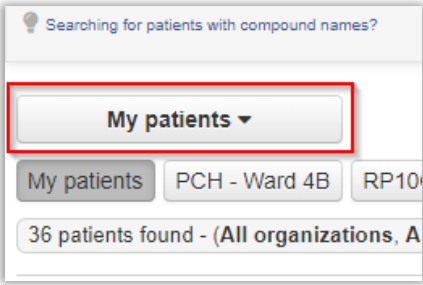
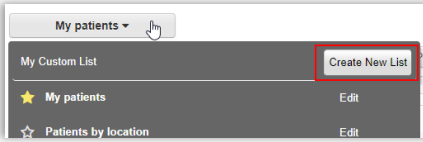
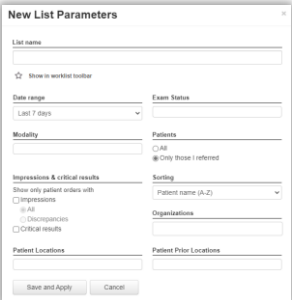
Viewports
Yellow outline indicates the selected viewport

Our purpose: We support our customers to provide excellent health care

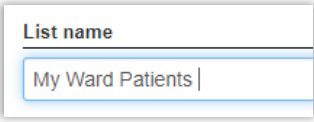
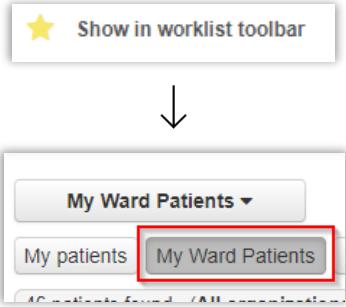

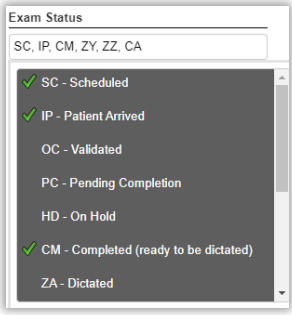
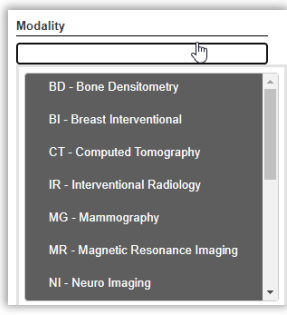
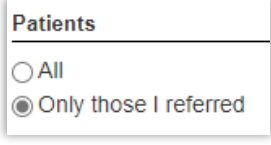
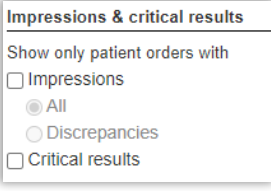
Creating a Worklist

Worklists allow you to create customised lists based on parameters that may be of interest to you. They can be tailored to broad or specific parameters. Some examples of this are:

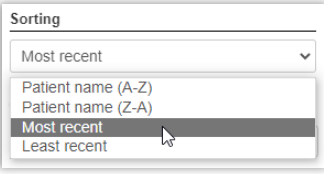
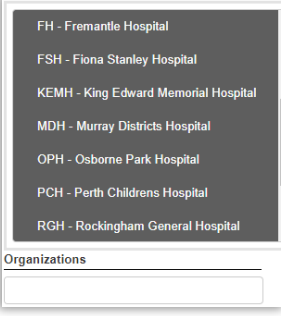

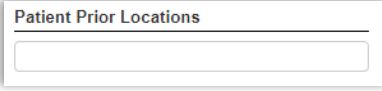
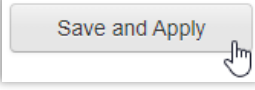
- **All SCGH ED Patients**
 - All SCGH Emergency with imaging in any order status.
- **My RPH ED Patients**
 - Only show RPH Emergency Department patients with imaging requests you submitted.
- **Patients with imaging – KEMH HDU**
 - Only show patients in the KEMH High Dependency Unit that have imaging available to review.
- **Scheduled Patients – OPH Antenatal**
 - Only show patients in the OPH Antenatal ward that have exams scheduled in Medical Imaging. The scheduled date and time of the exam will also be displayed.
- **Final Reports Available – Patients from PCH Ward 4B**
 - Only show patients in PCH Ward 4B that have final Medical Imaging reports available to review.

Screenshot	Instruction
	<p>Click on the 'Patients' tab.</p>
	<p>Click on the Worklist drop-down menu below the search bar.</p>
	<p>Click on 'Create New List.'</p>
	<p>'New List Parameters' dialog will appear.</p>

Our purpose: We support our customers to provide excellent health care

	<p>Give your list a name e.g. “My Ward Patients”</p>
	<p>Enable the ‘Show in worklist toolbar’ star to ‘favourite’ the worklist and have it appear on your ‘Worklist Toolbar’.</p>
	<p>Choose the date range of studies you are interested in.</p> <p>Note: You can choose time periods ahead of time to look for scheduled exams in Medical Imaging.</p>
	<p>Tick the appropriate ‘Exam/Order Status’ you wish to see in your Worklist.</p> <p><i>E.g. All exams that are currently Scheduled, In Procedure/Patient Arrived, Completed, a preliminary report is available, a final report is available.</i></p>
	<p>Choose the Modalities you wish to see.</p> <p>Note: Leave this field blank to see all modalities.</p>
	<p>Choose if you want to see only patients you have referred or all patients that match the other parameters.</p> <p>Note: Full name must be legibly written on the Medical Imaging request form for this to function.</p>
	<p>Tick the Critical Results box to only show exams with Critical Results.</p> <p>Note: Worklists with Critical Results included will display both Acknowledged and Unacknowledged results.</p>

Our purpose: We support our customers to provide excellent health care

	<p>Use the Sorting field to define how your results will be displayed.</p> <p>Select Most Recent to ensure the most recent examinations appear at the top of the list.</p>
	<p>Select the Organisations relevant to the worklist.</p> <p>Note: this will not filter 'Patient Location' options.</p>
	<p>Use the 'Patient Locations' menu to select any or all of the relevant wards. Select none to include all locations.</p> <p><i>e.g. I want to only see the patients from FSH wards FSEMER and FSEDOP.</i></p>
	<p>Note: 'Patient Prior Locations' is not in use in WA Health so should be left blank.</p>
	<p>Click 'Save and Apply' to complete the Worklist.</p>

For eLearning and further information, visit the MIRP hub:



<https://wahealthdept.sharepoint.com/sites/hss-customer-ict-mirp>

Our purpose: We support our customers to provide excellent health care