

Equipment

- ENT Ambu Screen
- Ambu Single Use Bronchoscope - Large bore suction
- Ribbed suction tubing - yellow connectors
- Lubricant
- Bronch safe - double swivel
- Sterile scissors
- 20 ml syringe
- Saline flush
- Flush channel spigot (located in Bronchoscope pack)

1

With sterile scissors, dilate the green flap on the Bronch Safe by inserting scissors and rotating 360 degrees.



2

Attach bronc safe to ETT, in-line suction must be removed.




3

Attach yellow ribbed suction tubing to the bronchoscope and wall suction.



4

Plug bronchoscope into Ambu screen and lubricate bronchoscope with lubricant spray.

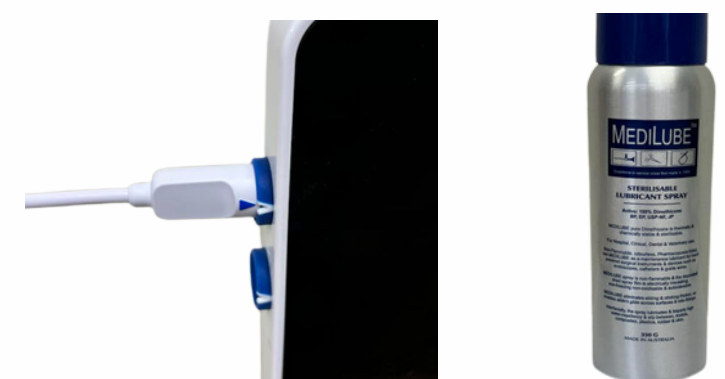

5

Should irrigation be required during bronchoscopy, insert irrigation spigot, located in the bronch pack, to top of Ambu scope and have a 20ml syringe and small bottle of WFI available.

